


**Subject: How to replace the Mirror unit on the OCTOPUS 900****To: Sales and Service Departments**

Date: 28.06.2007

### 1.1 Mirror Unit (1803300)

#### Removal

- Remove the front housing
- Remove the white cover for the mirror unit (1803158) by removing the 2 torx screws (screwdriver size 20). See figure 1.1.
- Unplug the ground cable and flat cables. See figure 1.2.
- Use an allen key screwdriver with ball-shaped head (size 5) to unscrew the 2 screws holding the mirror unit. See figure 1.3.
- Place the mirror unit on a clean surface in such a way that the mirror will not get scratched.

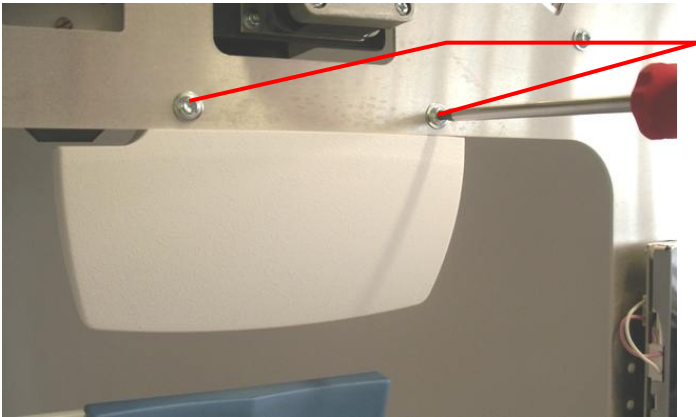


Remove the white cover for the mirror unit (1803158) by removing the 2 torx screws (screwdriver size 20)

Fig. 1.1



**WARNING:** When unscrewing the two screws you **MUST** hold the Mirror unit, so that it will **NOT** fall into the cupola!


**Subject: How to replace the Mirror unit on the OCTOPUS 900**

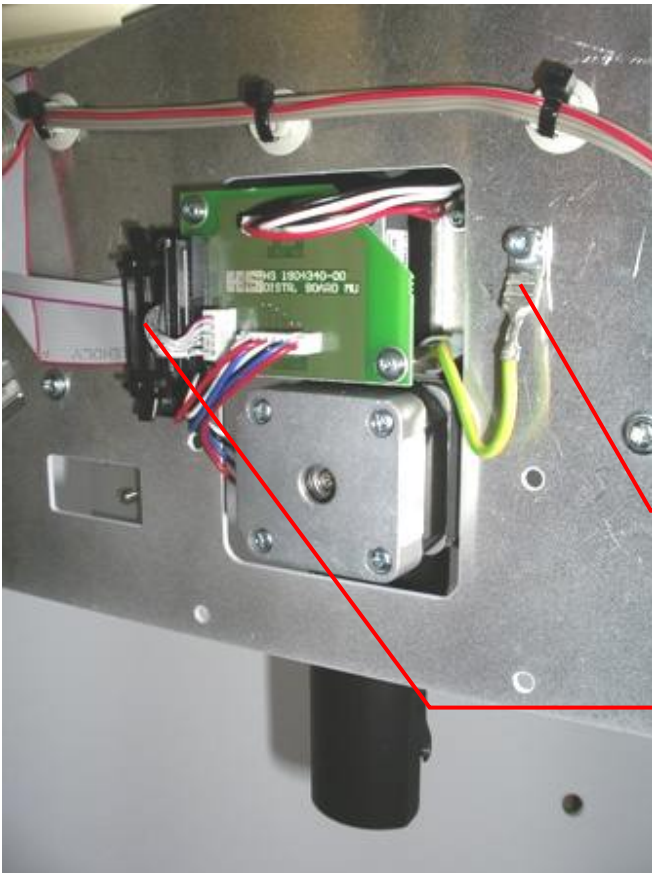

Unplug the ground cable

Unplug the flat cables

Fig. 1.2

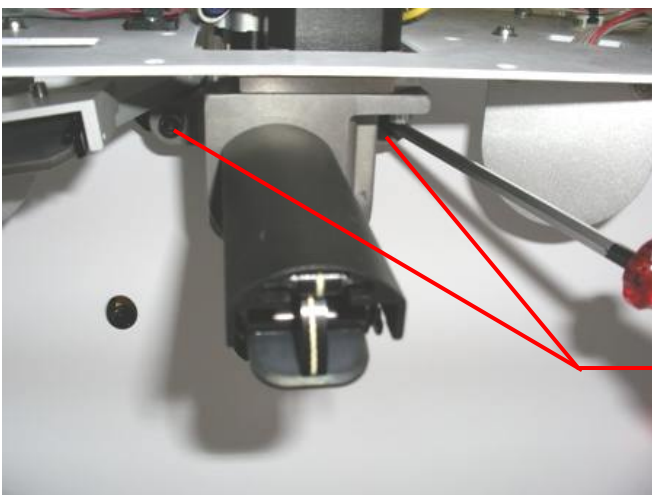

**NOTE: When removing these two screws you MUST hold the Mirror unit, so that it will NOT fall into the cupola!**

Use an allen key screwdriver with ball-shaped head (size 5) to unscrew the 2 screws holding the mirror unit.

Fig. 1.3

**Reassembly**

Reassemble in reverse order to removal.