

Clearing the Buffer of a 300 – 311 Octopus

The buffer (patient files) of the 301/311 will only hold 48 exams. In all software versions up to and including V5.15 when you try to input exam #49, the message “error 420 exam buffer full” will appear. In order to add this exam another needs to be removed.

In the current software V5.16 (used on the 300 Pro and Basic) no message will appear to warn you of a full buffer. Instead the patient preparation button becomes inactive and will not allow you to input another exam until delete an exam to create space

The following are the procedures to delete an exam.

9.5 Deleting Results

Select examination(s) in the examination file and press on the 'Edit' tab and the 'Delete' button. For your information the number of selected examinations is shown and the real deletion procedure must be started by pressing 'OK'. The procedure can not be rescinded.

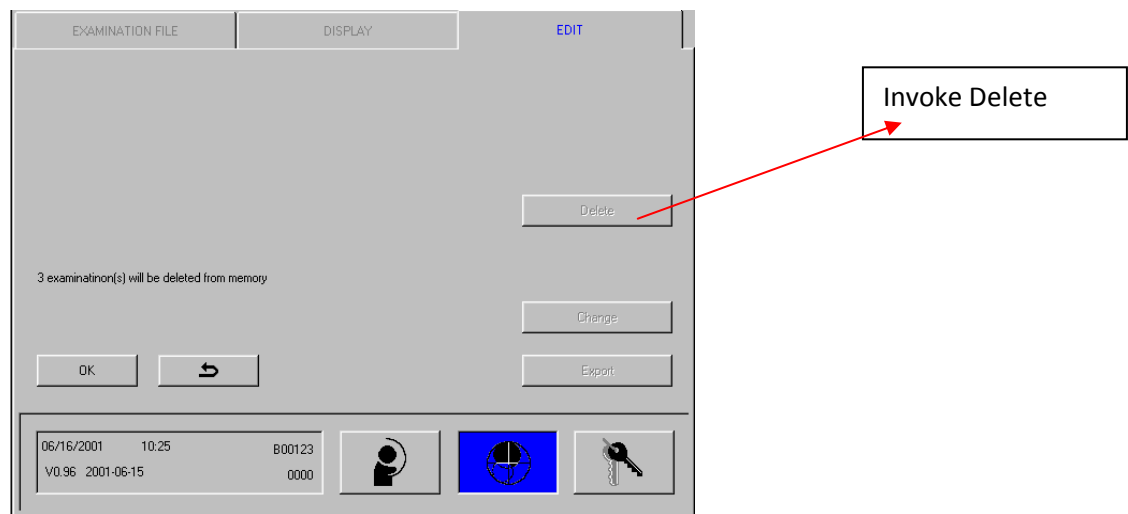


Figure 9-5: Processing – Edit – Delete

Deleting results

IMPORTANT: Delete examination results irretrievably from the data memory.