

## 7.15 Interface Board (1802224), Ethernet Interface Board (1802671)

### NOTE:

If you want to upgrade any older OCTOPUS 301 Perimeter to be able to use the Ethernet option (LAN Integration), then please refer to separate documentation given out by HAAG-STREIT or contact the OCTOPUS Service Department at HAAG-STREIT. The Ethernet interface board is already included in the OCTOPUS 300 and OCTOPUS 311 perimeters as from serial number 1841.

### Removal

- Remove 'Housing top' (see Chapter 7.1)
- Remove all the connectors from the 'Interface board'.
- Remove the two philips screws holding the 'Interface board'.

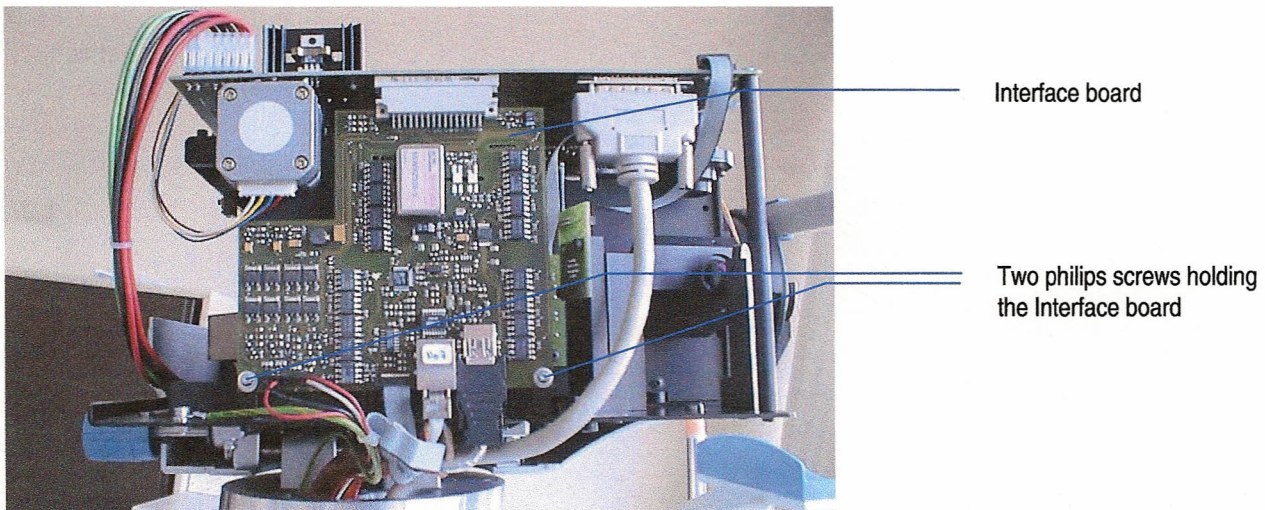


Figure 7-39

- Carefully pull the 'Interface board' down, out of the top connector (connector to the processor, 4 pin).
- Remove the 'Interface board' and store it in a static shielding bag.

### Reassembly

Reassemble in reverse order to removal.

### Check

After reassembly, enter the 'Service programs' and run the 'General test' to check that the instrument is fully functional.