

## REPLACING TOUCHSCREEN

### 7.9 Touch Screen Module (4-0692-0400)

The 'Touch screen module' is located towards the operator side of the instrument.

#### Removal

- Remove the two torx screws (size 20) on the bottom side of the 'Touch screen module'. See Figure 7-15.

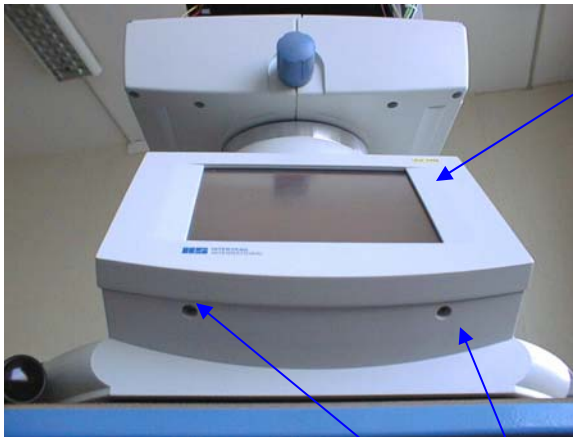


Figure 7-15

Touch screen module

Remove 2 torx screws (screwdriver size 20)

- Carefully lift up the bottom part of the 'Touch screen module' about 15mm, and then push from the top part of the 'Touch screen module' in a downward movement to be able to remove it from the stand.


Remove the hexagon socket screw (wrench size 3) holding the grounding cable to the 'Touch screen module', remove both the 2 pin and 50 pin connectors from the 'Touch screen module'.

- The 'Touch screen module' can now be removed completely. Place it on a clean, flat surface.

### Re-assembly

- In reverse order to removal.

**NOTE:** If it is difficult to slide in the top part of the 'Touch screen module' it is recommended to apply a little grease to the two rubber holding parts.

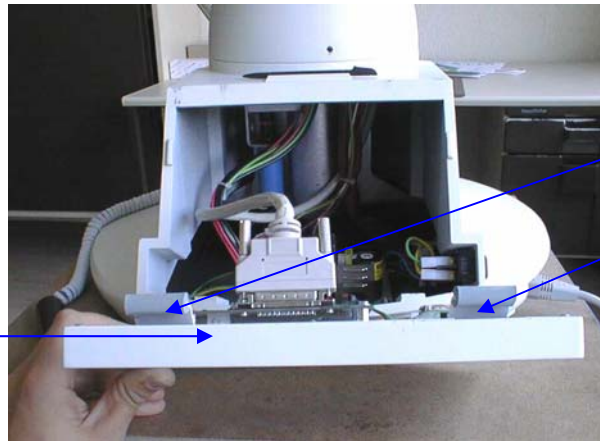

Apply a little grease to the two rubber holding parts to be able to re-assemble the 'Touch screen module' more easily.

Touch screen module

Figure 7-16

### Check

- Turn the power on, run the calibration and start an examination to check that the 'Touch screen module' works correctly.

**NOTE:** After a 'Touch screen module' has been replaced, calibration of the 'Touch panel' is required. Follow the procedure described in Chapter 8.2.1 '*Touch Screen Calibration*'.