


INSTRUCTIONS FOR REMOVING CHAIR TOP FX920

Removing the chair top.

1. Elevate chair base to highest point.
2. Remove base cover.
3. Remove upholstery attached to the chair with Velcro.
4. Remove the four Phillips screws (two on each side) going thru the plastic and into the seat board.
5. Tilt chair fully.
6. Locate two Phillips screws holding the seat board to the chair frame.
7. Remove seat board.
8. Return the chair to the upright position.
9. Disconnect all electrical connections and remove tie wraps.
10. Loosen set screw in round base nut.

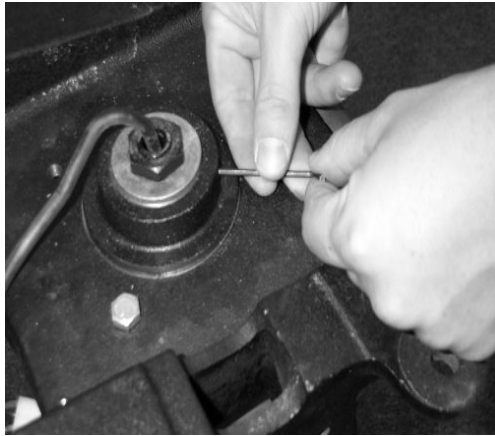

11. Turn the round base nut counterclockwise, and remove nut.



12. Remove chair top (two people are recommended).

**INSTRUCTIONS FOR REMOVING
CHAIR TOP FX920**