

INSTRUCTIONS FOR REPLACING SOLENOID VALVE

1. Lower chair completely.
2. Disconnect power cord.
3. Remove base covers (see separate instructions).
4. Follow the two black wires coming from the top of the solenoid valve assembly to a white plastic connector. Unplug white connector.
5. Loosen innermost screw holding valve bracket to base and remove outer screw in valve bracket.
6. Slide valve assembly outward to disengage bracket with mounting screw. Put rags or towels under valve assembly to catch any oil spilled.
7. Unscrew pressure line at bottom of valve and remove valve assembly.
8. Unscrew line coming from T-fitting to valve assembly.
9. Reconnect pressure line to new valve. Tighten fitting nut finger tight plus 1/3 turn.
10. Slide valve bracket under lock washer on innermost mounting screw. Replace outer mounting screw and tighten both screws.
11. Reconnect plug on valve wiring, plug power cord into receptacle and check operation of base.
12. Replace motor cover.

